

Detection within 10 min. | High sensitivity and specificity | Competitive assay


Please read the manual carefully before using this test kit.

**BAROS-N1C Kit** is a rapid drug test kit that identifies tetrahydrocannabinol(THC), cocaine cocaine(COC), opium morphine(MOR/OPI), zolpidem(ZOL), methamphetamine(MET), ketamine(KET) and other drugs in suspected samples. This test kit uses a specific antibody of each drug to provide excellent detection sensitivity. In addition, the results can be checked in less than 10 minutes and can be easily determined.

**Product Package**



**BAROS-N1 User Guide**

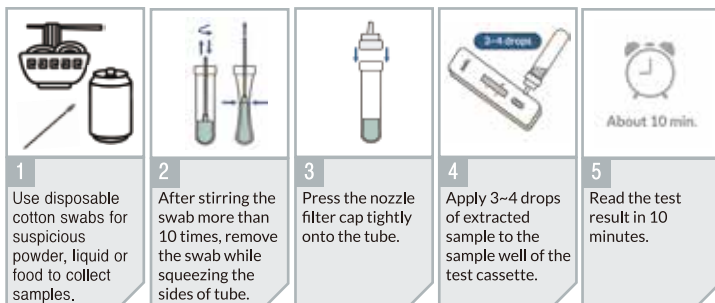

**Interpretation of test result**

The BAROS-N1 detection results are as follows.

It consists of control and test line, and depending on the presence or absence of drug, it is read as follows.

<b>Positive</b>		<b>When test line is disappeared</b> Higher drug exists more than LOD. There are harmful amounts of drug for human
<b>Negative</b>		<b>When test line and control line are colored</b> Don't worry, the drug doesn't exist.
<b>Invalid</b>		<b>When control line is disappeared</b> The test is not performed normally. Retest is required.

※ The result after 30 minutes are unreliable.

**Caution**



1. This product cannot be used for any purpose other than to detect drug.
2. Keep out of reach of children and infants.
3. Check the use-by date, damage or deterioration before using.
4. Conduct the test on a flat surface, and hold the buffer tube vertically and drop it slowly.
5. Results after 30 minutes are unreliable.
6. Look closely at the test cassette in a bright place.
7. People who have difficulty distinguishing colors or have low vision are difficult to check results.