

Detection within 10 min. | High sensitivity and specificity | Competitive assay


Please read the manual carefully before using this test kit.

BAROS-N6 6 Multi Drug Test Kit is a kit that detects 6 drugs or psychotropic drugs. This detection kit uses a specific antibody of each drug to provide excellent detection sensitivity. In addition, the method of use is simple and the results can be checked in less than 10 minutes.

Product Package





BAROS-N6 User Guide

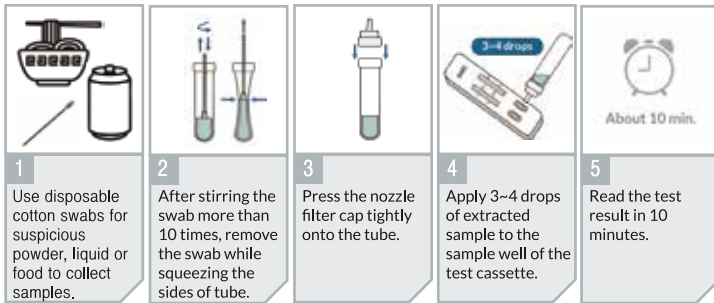

Interpretation of test result

The BAROS-N6 detection results are as follows.  
 It consists of control(C) and test spots(1, 2, 3, 4, 5, 6).  
 About the presence or absence of each target drug, read as below.

When control line is disappeared  
 The test is not performed normally.  
 Retest is required.

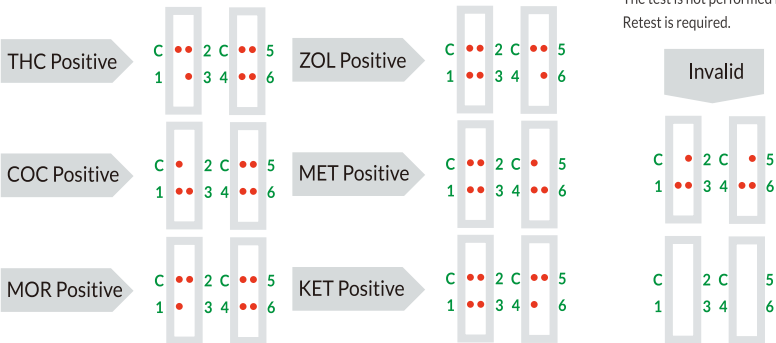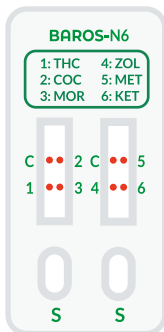

\* The result after 30 minutes are unreliable.

Caution



1. This product cannot be used for any purpose other than to detect drug.
2. Keep out of reach of children and infants.
3. Check the use-by date, damage or deterioration before using.
4. Conduct the test on a flat surface, and hold the buffer tube vertically and drop it slowly.
5. Results after 30 minutes are unreliable.
6. Look closely at the test cassette in a bright place.
7. People who have difficulty distinguishing colors or have low vision are difficult to check results.